




EQUIPMENT SPECIFICATIONS





# CyberKnife® Robotic Radiosurgery System

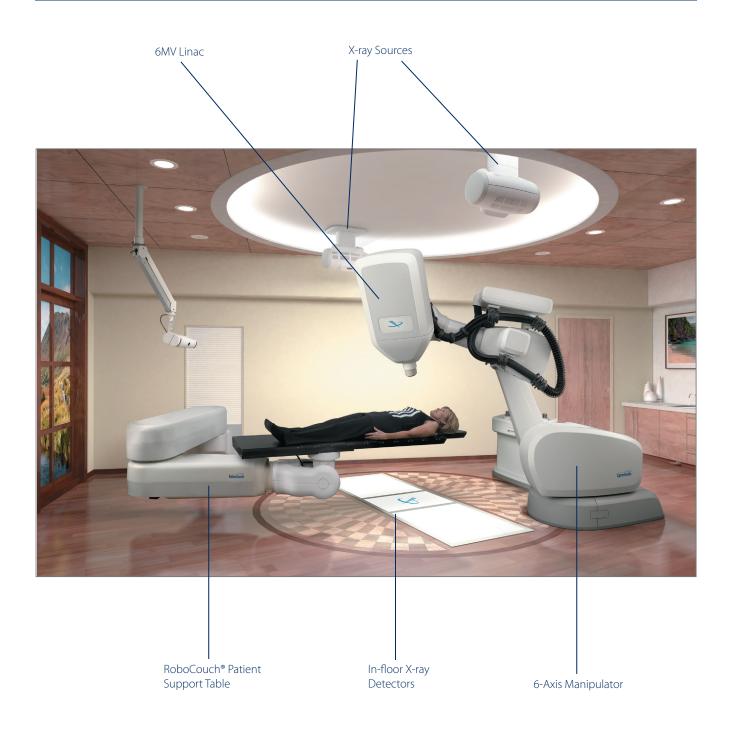



# TABLE OF CONTENTS

1		STEM OVERVIEW	
2		EATMENT DELIVERY	
	2.1	LINEAR ACCELERATOR (Linac).	4
		2.1.1 X-ray Energy	
		2.1.2 Field Size	
		2.1.3 Dose Rate.	
		2.1.4 Flatness.	
		2.1.5 Asymmetry	
		2.1.6 Leakage         2.1.7 Penumbra	
		2.1.8 Collimator Transmission.	
		2.1.9 Dosimetry System	
		2.1.10 Absolute Dose Calibration	
		2.1.11 Field Geometry	
	2.2	SIX-AXIS ROBOTIC MANIPULATOR.	
		2.2.1 Workspace	
		2.2.2 Safety Zones	8
	2.3	SECONDARY COLLIMATOR SIZES	
		2.3.1 Fixed Collimators.	
		2.3.2 Iris™Variable Aperture Collimator	
		2.3.3 Xchange® Robotic Collimator Changer	
	2.4	TREATMENT DELIVERY SYSTEM CONTROL CONSOLE HARDWARE SPECIFICATION	
3	IMA	AGING	11
	3.1	X-RAY GENERATORS	. 12
	3.2	X-RAY SOURCES	. 12
	3.3	X-RAY DETECTORS.	. 12
4	TAR	RGET TRACKING	13
	4.1	SYNCHRONY® RESPIRATORY TRACKING SYSTEM	. 14
	4.2	INTEMPO™ ADAPTIVE IMAGING SYSTEM	. 15
	4.3	FIDUCIAL TRACKING	. 16
	4.4	XSIGHT® SPINE TRACKING SYSTEM	. 18
	4.5	XSIGHT LUNG TRACKING SYSTEM	. 18
	4.6	6D SKULL TRACKING	. 19
	4.7	SUPPORTED TRACKING COMBINATIONS	. 19
5	TRE	EATMENT PLANNING	19
	5.1	HARDWARE SPECIFICATION.	. 19
	5.2	MULTIPLAN® SYSTEM SOFTWARE	. 20
	5.3	DISPLAY LAYOUTS AND CONTROLS	. 20
	5.4	USER PRIVILEGE AND SECURITY SETTINGS	. 20
	5.5	FUSE (IMAGE REGISTRATION)	. 20
	5.6	CONTOUR (DELINEATION)	. 21
		5.6.1 VOI (Volume of Interest)	. 21
		5.6.2 Skin	. 21
		5.6.3 Spine Tracking Volume	. 21
		5.6.4 Ball-cube	. 21
	5.7	TREATMENT PLANNING.	. 22
		5.7.1 Isocentric Planning	. 22
		5.7.2 Conformal Planning	. 22
		5.7.3 Dose Calculation Algorithms	. 22
	5.8	EVALUATION	. 22
		5.8.1 Plan Statistics	. 22
		5.8.2 Isodose Curves	. 22
		5.8.3 3D Rendering	. 23

# CyberKnife<sup>\*</sup>

	5.9 QA	A TOOLS	. 23
	5.9	9.1 Plan Comparison	23
	5.9	9.2 Plan Summation	23
	5.9	9.3 Beam List	23
	5.9	9.4 Animation	23
	5.10 CC	DMMISSIONING TOOLS	23
	5.1	10.1 Graphical Display	23
	5.11 M	ONTE CARLO DOSE CALCULATION	24
	5.12 40	O TREATMENT OPTIMIZATION AND PLANNING (OPTIONAL).	25
	5.13 SE	EQUENTIAL OPTIMIZATION (OPTIONAL)	25
	5.14 MI	ULTIPLAN® MD SUITE (OPTIONAL)	26
		ULTIPLAN QUICK REVIEW (OPTIONAL)	
		ENSITY OVERRIDE	
		UICKPLAN <sup>TM</sup> (OPTIONAL)	
		JTOSEGMENTATION™ (OPTIONAL).	
		REATMENT TIME REDUCTION TOOL	
		MPROVED PET SUPPORT.	
6		MANAGEMENT	
Ĭ		YBERKNIFE® DATA MANAGEMENT SYSTEM	
		1.1 Data Server	
		1.2 Administrative Workstation	
		IPAA COMPLIANCE SUPPORT	
7		IT POSITIONING SUPPORT	
′		REATMENT COUCH	
	7.1 111	Standard Treatment Couch	
	7.1.1		
		REATMENT TABLE TOP	
	7.2 11		
	7.2.1	·	
0		ATORY CLASSIFICATION	
8	REGUL	ATORT CLASSIFICATION	. ၁၁
	6.7.0	5 TABLES	
		F TABLES	
		Robotic Manipulator Specifications	
		ris™Variable Aperture Collimator Specifications	
		Kchange® Robotic Collimator Changer Specifications	
		K-ray Generator Specifications	
		K-ray Sources Specifications.	
		K-ray Detector Specifications	
		T Requirements for Target Tracking	
		reatment Correction Ranges	
		argeting Accuracy with Various Targeting Methods	
		Synchrony® Respiratory Tracking System Specifications	
		nTempo™ System Features	
		iducial Tracking Specifications.	
		Approved Fiducials for Soft Tissue Tracking	
Ta	<b>ble 14:</b> A	Approved Fiducials for Bone or Spine Implantation	. 17
Ta	ble 15: X	(sight® Spine Tracking System Features	. 18
Ta	ble 16: X	(sight Lung Tracking System Features	. 18
Ta	<b>ble 17:</b> 6	5D Skull Tracking System Features	. 19
Ta	<b>ble 18:</b> S	Selectable (YES) and Non-Selectable (NO) Tracking Mode/Patient Position Combinations	. 19
Ta	<b>ble 19:</b> P	Patient Positioning System Specifications	. 30
Ta	ble 20: ⊤	reatment Couch Top Specifications	32



#### 1. SYSTEM OVERVIEW

The Accuray CyberKnife® Robotic Radiosurgery System consists of the following major functional sub-systems:

- Treatment Delivery
- Imaging

Target Tracking

• Treatment Planning

- Data Management
- Patient Support



CyberKnife System

This specification document includes components of both the CyberKnife Robotic Radiosurgery System as well as the CyberKnife VSI™ System. Contact a local Accuray representative for details on the products and configurations that are available in your region. This document provides a technical description and manufacturer's specifications of the CyberKnife System. It is not intended as a substitute for clinical training or to provide guidelines for clinical use.

#### **System Targeting Accuracy**

System targeting accuracy accounts for the inaccuracies in all components of the system as it applies to a particular treatment delivery modality. This may be understood as a combination of mechanical, targeting, tracking and clinical accuracy and includes error sources from the CT scan, treatment planning, patient tracking and dose delivery systems. All these elements comprise the clinically relevant accuracy which may be termed the overall average CyberKnife error.



The CyberKnife VSI System shown with the RoboCouch Patient Positioning System (optional)

The overall average CyberKnife System error is less than 0.95 mm RMS when CT slice spacing of 1.25 mm or less is used. When used with the Synchrony® Respiratory Tracking option, the overall average CyberKnife System error is less than 1.5 mm RMS for CT slice spacing of 1.25 mm or less.



#### 2. TREATMENT DELIVERY

The CyberKnife® treatment delivery system is comprised of an X-band linear accelerator mounted on a 6 degree-of-freedom (DOF) robotic manipulator. An imaging system consisting of two kV X-ray sources and detectors takes images of the patient throughout the treatment.

#### 2.1.1 X-Ray Energy

The X-ray energy is 6 MV nominal (photon). The quality index ( $TPR_{20/10}$ ) is 0.62 to 0.67 for a field size of 60 mm diameter with a source-to-axis distance (SAD) of 800 mm with an ion chamber. The depth of maximum dose,  $D_{max}$ , is 15 mm  $\pm$  2mm for a fixed field size of 40 mm.

#### 2.1.2 Field Size

Secondary collimators are provided to produce circular fields at a reference treatment distance of 800 mm source-axis distance (SAD) with nominal diameters of 5, 7.5, 10, 12.5, 15, 20, 25, 30, 35, 40, 50 and 60 mm. There are two types of secondary collimators available; fixed aperture collimators and the Iris $^{\text{TM}}$  Variable Aperture Collimator (see Section 2.3.2).

#### 2.1.3 Dose Rate<sup>2</sup>

Depending on territory, the CyberKnife System is equipped with a linac operating at one of the following nominal dose rates

- a) 1000 Monitor Units per minute (MU/min)
- b) 800 MU/min
- c) 600 MU/min
- d) 400 MU/min

The nominal dose rate is measured at a SAD of 800 mm and at a depth of 15mm in a water phantom for a field size of 60 mm diameter. Dose rate is measured with a standard calibrated ion chamber of 0.6 cm<sup>3</sup> volume or less, using the 60 mm fixed aperture collimator.

#### 2.1.4 Flatness

The beam flatness measured along two orthogonal axes shall not exceed 14% when measured using a stereotactic diode in a water phantom at an SAD of 800 mm and a depth of 50 mm with a 40 mm secondary collimator.

The flatness is defined as the maximum variation in the profile within the central 80% region of the FWHM. This is reported as the percentage difference between the maximum value (100% by definition) and the minimum value within this region.

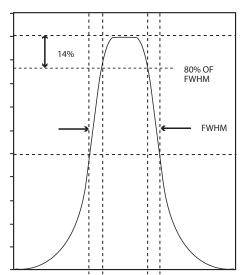


Figure 1. Illustration of the regions of dose measurement that are used in the flatness calculation.

<sup>&</sup>lt;sup>1</sup> See Accuray Physics Essentials Guide, P/N 029577.A.

<sup>&</sup>lt;sup>2</sup> All presented configurations may not be available in all territories. Please consult the local Accuray representative for more information.



#### 2.1.5 Asymmetry

Photon therapy beam asymmetry is the percent difference in the area on the right and left sides of the dose profile curve divided by the sum of the left and right areas under the dose profile curve within the central 50% field width. Asymmetry is less than 2%. This measurement is made in a water phantom for the collimator field size of 40 mm diameter at SAD of 800 mm and at a depth of 50 mm with a stereotactic diode.

Formula for asymmetry calculation:  $\frac{|a-b|}{a+b} \times 100\%$ 

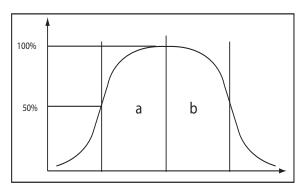


Figure 2. Illustration of the regions of dose measurement that are used in the asymmetry calculation.

#### 2.1.6 Leakage

The leakage dose (measured anywhere in the patient plane outside of the maximum useful beam) is less than 0.1% of the absorbed dose at the reference treatment distance of 800 mm SAD. In all other directions, the absorbed dose 1 m from the path of the electrons between the electron gun and the target is less than 0.1% of the absorbed dose at the reference treatment distance.

The patient plane is defined as a plane circular surface of radius 2 m centered on and perpendicular to the axis of the beam at the reference treatment distance. The X-ray measurements may be averaged over an area not to exceed 100 mm<sup>2</sup>.

#### 2.1.7 Penumbra

The radial distance between the 80% and 20% intensity points will not exceed 4.5 mm with a 40 mm fixed secondary collimator. Measurements shall be taken with a stereotactic diode detector in a water phantom at 50 mm depth and at SAD of 800 mm, or by using film with the same beam and an equivalent depth in a phantom.

#### 2.1.8 Collimator Transmission

The X-ray transmission through the secondary collimator absorber (no hole) shall not exceed 1% compared to the dose rate with a 60 mm collimator when the nominal dose rate is used.

#### 2.1.9 Dosimetry System

A two-channel primary/secondary dosimetry system is provided. The following specifications apply for both of the dosimetry channels:

#### - Reproducibility with Linac Orientation

Reproducibility of the dose delivered with the different orientations of the linac is less than  $\pm$  2% compared to the Beam Down orientation when 100 MU is delivered.

#### Linearity


The linearity shall be within ±1% or 1 MU, whichever is greater, of accumulated doses between 10 and 1,000 MU.

#### 2.1.10 Absolute Dose Calibration

An MU is nominally one centiGray delivered to a tissue-equivalent material at a depth of 15 mm and 800 mm SAD with a 60 mm fixed aperture collimator.



# 2.1.11 Field Geometry

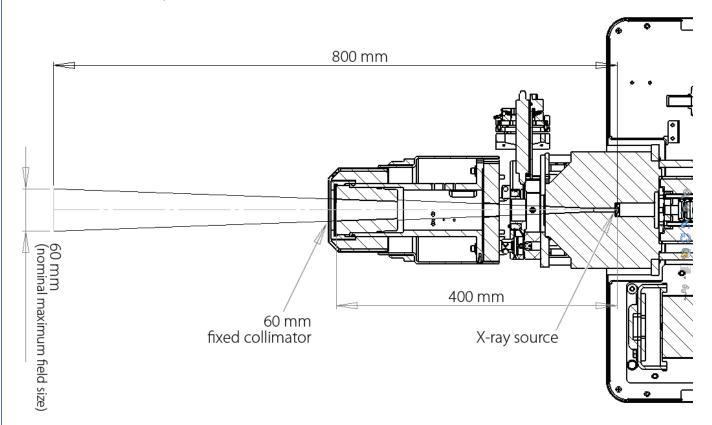

# 2.2 SIX-AXIS ROBOTIC MANIPULATOR


The CyberKnife® System utilizes a 6-DOF robotic manipulator to position the linac in space and deliver dose. Relevant specifications of the manipulator are provided in Table 1.

Table 1: Robotic Manipulator Specifications

Six-Axis Manipulator Specifications (Selected) <sup>3</sup>			
<b>Payload</b> 240 kg (528 lb)			
Maximum Reach	3100 mm (122 in)		
Number of Axes	6		
Repeatability	< ± 0.12 mm (< ± .004 in)		
Footprint <sup>4</sup>	1006 mm (39.61 in) x 1006 mm (39.61 in)		
Work Envelope	55 m³ (1942 ft³)		
Weight	1267 kg (2793 lb)		

 $<sup>^3</sup>$ For detailed specifications of the configuration available in respective territories, please contact the local Accuray representative.

 $<sup>^4</sup>$ For detailed installation requirements, please contact the local Accuray representative.

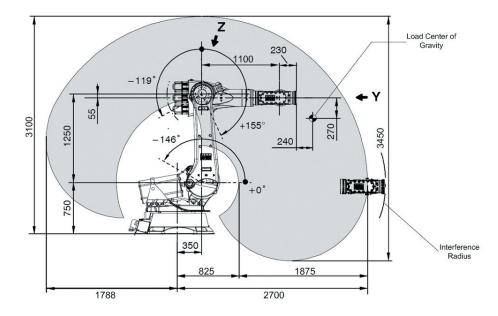


The following diagram shows the volume that the robotic manipulator is capable of moving in.

The robotic manipulator is programmed to move in a fixed and pre-determined workspace. The designed workspace accounts for the positions of objects in the treatment suite, including the treatment couch, the imaging sources and detectors, the floor and the ceiling, and eliminates collision hazards by creating suitable paths for the robotic manipulator to move in. Additionally, the workspace is comprised of pre-assigned points in space, termed nodes, where the manipulator is allowed to stop in order to deliver radiation dose. At each node, the linac can deliver radiation from multiple beam angles. It may be noted that the representation above is conceptual as the workspace and the treatment paths adopted by the robotic manipulator are dependant on the location of the target and patient anatomy being treated.


#### 2.2.1 Workspace

# Robot Workspace

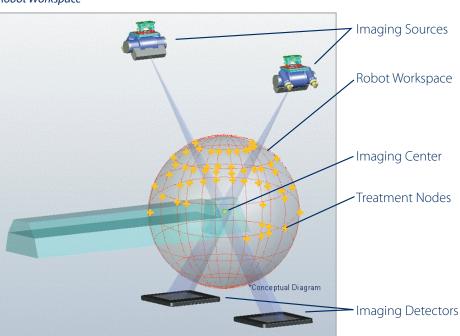



# 2.2.2 Safety Zones

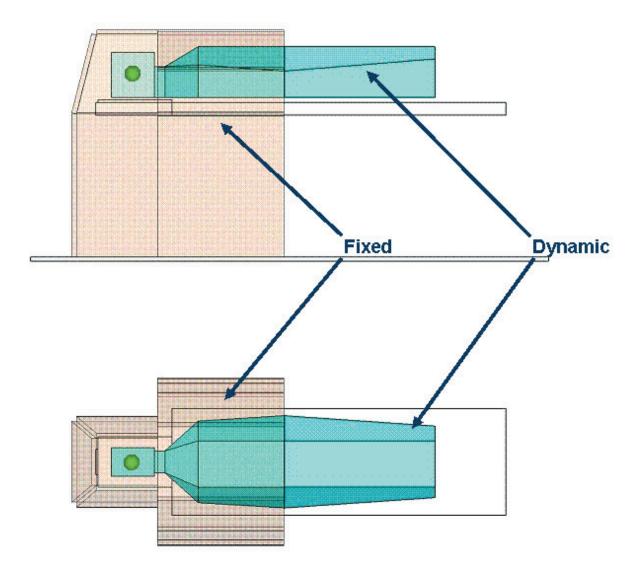

The robot workspace also takes into consideration the position of the patient and is designed to avoid contact with the patient. This is achieved by creation of a safety zone around the patient and the treatment couch. As shown in the representation above, the safety zone consists of two elements: fixed and dynamic. The fixed safety zone is rigidly attached to the imaging center and thereby the part of the patient body being treated while the dynamic safety zone is designed to encompass the entire patient body and always lies within the fixed safety zone. The size of the dynamic safety zone is user selectable based on individual patient sizes (Small, Medium or Large).

Patient safety is further enhanced by the presence of a contact detection sensor at the distal end of the secondary collimator housing on the linac. Contact with the sensor causes an Emergency Stop (E-STOP) condition halting all motion of the system.




#### 2.3 SECONDARY COLLIMATOR SIZES

The CyberKnife® System uses multiple secondary collimator sizes that can be changed manually or automatically (optional with the Xchange® Robotic Collimator Changer) to deliver beams as defined by the treatment plan. Also available, optionally, is the Iris™ Variable Aperture Collimator which allows the collimation aperture to automatically change for each beam, if necessary.

#### 2.3.1 **Fixed Collimators**

The CyberKnife System is supplied with fixed secondary collimators delivering circular field sizes of 5, 7.5, 10, 12.5, 15, 20, 25, 30, 35, 40, 50 and 60 mm diameter at 800 mm SAD. These collimators can be changed to vary the beam size as generated by the treatment plan. For each fixed collimator, the manipulator traverses a separate path.

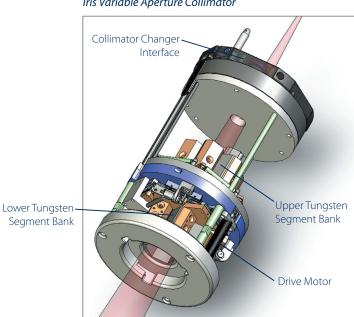

#### Iris Variable Aperture Collimator

#### Iris Variable Aperture Collimator<sup>5</sup>

The Iris Variable Aperture Collimator creates beams with characteristics virtually identical to those of fixed collimators. It consists of two banks of 6 tungsten segments each with each bank creating a hexagonal aperture. The two are offset by 30° relative to each other resulting in a dodecahedral (12-sided) aperture when viewed from one end of the collimator to the other. The Iris Variable Aperture Collimator replicates the existing 12 fixed collimator sizes.

**Table 2: Iris Variable Aperture Collimator Specifications** 

Iris™ Variable Aperture Collimator Specifications				
Circularity	• The standard deviation of the radial distance from the beam axis to the 50% dose level is less than 2% of the average radial distance			
Collimator Transmission Dose Rate	<ul> <li>Maximum: &lt; 0.2% of the delivered dose rate</li> <li>Average: &lt; 0.1% of the delivered dose rate</li> </ul>			
Reproducibility	Mechanical: less than 0.1 mm     Treatment field size: < 0.2 mm at the nominal treatment distance of 800 mm			
Available Apertures	• Effective collimation sizes: 5, 7.5, 10, 12.5, 15, 20, 25, 30, 35, 40, 50 and 60 mm field sizes at 800 mm SAD			



#### 2.3.3 Xchange® Robotic Collimator Changer

The Xchange® Robotic Collimator Changer automatically changes collimators before and during patient treatment. This eliminates the need to stop treatment and enter the room minimizing patient disturbance and possible need for adjustment to patient setup. The Xchange System is compatible with all the fixed secondary collimators provided by Accuray and the Iris™ Variable Aperture Collimator.



Xchange Robotic Collimator Changer

# Table 3: Xchange Robotic Colllimator Changer Specifications

Xchange System Specifi	Xchange System Specifications				
Secondary Collimators	<ul> <li>Accommodates two secondary collimator housings, one for Fixed Collimators, and one for the Iris Variable         Aperture Collimator</li> <li>12 Fixed Collimators</li> <li>Iris Variable Aperture Collimator</li> </ul>				
Iris Variable Aperture Collimator	Stored in individual housing receptacle in Xchange Collimator table				
Instrumentation	<ul> <li>Presence sensor for all collimators in respective collimator cup and housing receptacle</li> <li>Laser alignment sensors in each cup</li> <li>Three Xchange calibration Iso-posts</li> <li>Safety sensors located in each receptacle</li> </ul>				

#### 2.4 TREATMENT DELIVERY SYSTEM CONTROL CONSOLE HARDWARE SPECIFICATION

- Dual processor with each processor having a clock speed  $> 2~\mathrm{GHz}$
- 4 GB of memory
- Set of SAS 15K RPM disk drives in a RAID 0 (Mirrored) configuration providing approximately 130 GB of storage
- Two Gigabit Network Interface Cards (NIC)
- Redundant power supplies




#### 3. IMAGING

The CyberKnife® System employs kV X-ray imaging to provide target localization during treatment. The imaging system consists of two X-ray sources mounted to the ceiling, and corresponding image detectors mounted in the floor. The X-ray sources are positioned such that the generated beams intersect orthogonally and create an imaging center located 92 cm (36.22 in) from the floor. All treatments on the CyberKnife System are based around the imaging field of view.

The live images are digitized and compared to images synthesized from the patient's CT data (Digitally Reconstructed Radiograph, or DRR). This technique allows for determination of intra-fraction target shifts and automatic compensation by the treatment manipulator during treatment delivery. See Table 8 for the ranges of motion that can be compensated for without moving the patient. When used with the Synchrony® Respiratory Tracking System, the system can compensate for target translation over a range of ±25 mm in any direction.

#### Typical Imaging System Set-up

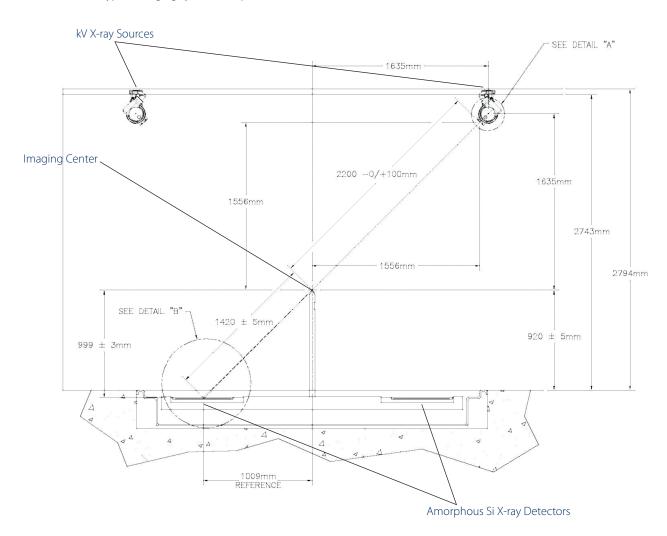

 $<sup>^6</sup>$ When used with the RoboCouch System, the range is  $\pm 1.5^\circ$  for all rotations.



# 3.1 X-RAY GENERATORS

Dependant on system configuration, one from amongst the following X-ray Generators is supplied with the CyberKnife® System.

Table 4: X-ray Generator Specifications

	Tall X-ray Generators	Compact X-ray Generators
CONSTANT POTENTIAL POWER RATING (KW)	37.5	50.0
RADIOGRAPHIC kVp RANGE	40 - 120 ± (5% + 1 kVp)	40 - 150 ± (5% + 1 kVp)
RESOLUTION	1 kVp	1 kVp
ma range and stations	25, 50, 75, 100, 150, 200, 250, 300 ± (5% + 1 mA)	10, 12.5, 16, 20, 25, 32, 40, 50, 64, 80, 100, 125, 160, 200, 250, 320, 400, 500, 640 ± (5% + 1 mA)
POWER OUTPUT	300 mA @ 125 kVp	640 mA @ 78 kVp, 500 mA @ 100 kVp, 400 mA @ 125 kVp, 320 mA @ 150 kVp
EXPOSURE TIME	0.001 – 5 seconds ± (10% + 1-100ms)	0.001 - 10 seconds ± (1% + 0.1 ms)
mAS	0.1 – 550 mAs	0.1 – 640 mAs

# 3.2 X-RAY SOURCES

Table 5: X-ray Sources Specifications

, ,		
X-Ray Sources Specifications <sup>7</sup>		
Electrical		
- Circuit	3-phase	
- Nominal tube voltage	40 – 150 kV	
- Nominal focal spot value	Large focus: 1.2 mm Small focus: 0.6 mm	
- Nominal Anode input power	Large focus: 100 kW Small focus: 40 kW	
Aluminum Filter	2.5 mm	
Firing Modes	Synchronous and Asynchronous	
Collimator Type	Fixed Aperture	

# 3.3 X-RAY DETECTORS

One pair of X-ray Image Detectors is supplied with the CyberKnife System.

**Table 6: X-ray Sources Specifications** 

X-Ray Imaging Detectors Specifications		
Number of Pixels	1024 x 1024	
Number of Active Pixels	1012 x 1012	
Pitch	400 μm	
Total Area	409.6 x 409.6 mm <sup>2</sup>	
Integration Time (minimum)	66.45 ms	
Dynamic Range	> 75 dB	
Radiation Energy	40 keV – 15 MeV	
Detector Housing	672 x 599 x 44 mm <sup>3</sup>	



## 4. TARGET TRACKING<sup>8</sup>

Accurate target tracking and compensating for target motion are an integral part of the CyberKnife® System and its capabilities. The target is tracked throughout the entire treatment and delivery is automatically altered to compensate for any motion.

CT requirements to achieve published accuracy specifications for all tracking systems are listed in Table 7.

Table 7: CT Requirements for Target Tracking

Maximum Slices	512
kVp	120
mAs	Scanner Maximum (minimum 400)
Slice Thickness	Contiguous slices (no gaps); < 1.5 mm slice thickness

Target tracking and motion compensation are achieved through the use of the imaging system integrated with the treatment delivery system. The image guidance system calculates the required adjustment to the patient's position. During patient set-up, the operator has the option to allow the system to automatically align the patient or to manually adjust the patient on the treatment table, as needed. This process aligns the CT origin close to the imaging center and is required at the time of patient setup only. During the treatment, the system automatically corrects the linac position (and thus radiation targeting) for any patient motions within a specified tolerance (see Table 8 for details).

**Table 8: Treatment Correction Ranges** 

	RoboCouch® System	RoboCouch System with Prostate Path	Standard Treatment Couch	Standard Treatment Couch with Prostate Path
X,Y, and Z	± 10 mm	± 10 mm	± 10 mm	± 10 mm
X,Y, and Z with Synchrony® Respiratory Tracking System	± 25 mm	± 25 mm	± 25 mm	± 25 mm
Pitch	± 1.5°	± 5°	± 1°	± 5°
Roll	± 1.5°	± 1.5°	± 1°	± 1°
Yaw	± 1.5°	± 1.5°	±3°	± 3°

Target tracking on the CyberKnife System is achieved by the use of one or more of the available tracking methods namely fiducial tracking, the Xsight® Spine Tracking System, the Xsight Lung Tracking System and 6D skull tracking. The Synchrony® Respiratory Tracking System and the InTempo™ Adaptive Imaging System, as described in later sections of this document, are motion compensation technologies and are used in conjunction with applicable tracking methods. The Synchrony System is used to compensate for repetitive motion such as that induced by breathing. The InTempo System is used to compensate for rapid, erratic motion and other patterns seen in the prostate.

Accuracy specifications for the various tracking methods are detailed in Table 9.

**Table 9: Targeting Accuracy with Various Targeting Methods** 

	Without Synchrony Respiratory Tracking System	With Synchrony Respiratory Tracking System
Fiducial Tracking  Xsight Lung Tracking System <sup>9</sup>	Better than 0.95 mm RMS	Better than 1.5 mm RMS
Xsight Spine Tracking System  6D Skull Tracking	Better than 0.95 mm RMS	N/A

<sup>&</sup>lt;sup>8</sup>All presented options may not be available in every territory. Please contact the local Accuray representative to learn more.


<sup>9</sup>Rotational targeting accuracy for the Xsight Lung Tracking System is calculated by the Xsight Spine Tracking System.



#### 4.1 SYNCHRONY® RESPIRATORY TRACKING SYSTEM


The Synchrony® Respiratory Tracking System continuously synchronizes treatment beam delivery to the motion of a target that is moving with respiration. The system operates by creating a correlation model between the patient's breathing pattern, monitored in real-time, and the location of the target at various points in the respiration cycle. The location of the target is determined by using X-ray imaging to visualize the lesion or internal markers (fiducials), while the breathing pattern is tracked and monitored using external markers (LED-based, fiber optic tracking markers) in real-time.

X-ray images required for building the correlation model between external chest wall motion and internal target motion may be acquired manually, in a User Defined sequence or by using the fully Automatic functionality available. The system automatically determines the best correlation model type to be utilized for the particular treatment by choosing the model type that minimizes overall correlation error. The model is chosen from linear, curvilinear and bi-curvilinear forms. The model is based on the latest 15 sets of X-ray images taken and is updated every time a new image is taken with the oldest image being discarded. The allowable error in the correlation model can be modified by the user and the violation of these set limits causes a pause in the treatment delivery.

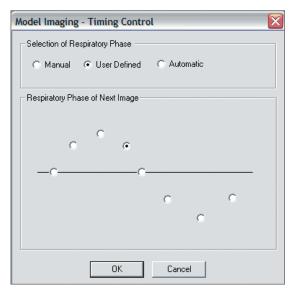

Building the Correlation Model

#### User Interface showing the Respiratory Model, Correlation Graphs and Correlation Error

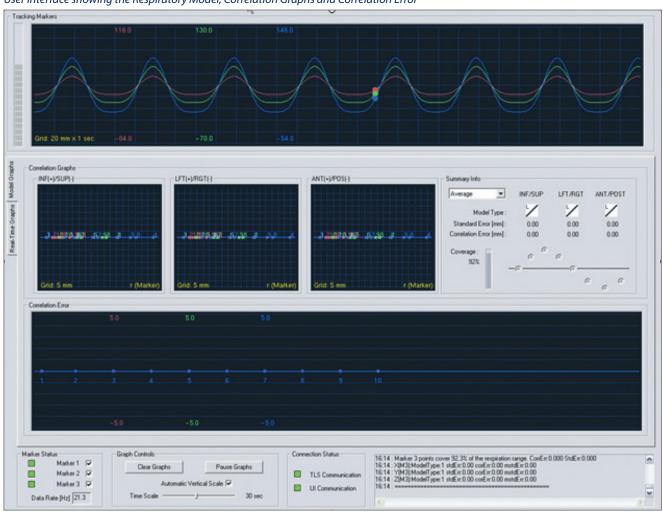

**Table 10: Synchrony Respiratory Tracking System Specifications** 

Synchrony® Respirator	y Tracking System	Specifications
Tracking		<ul> <li>Accomplished by the use of LED based, fiber optic markers</li> <li>3 markers (ideal); 1 marker (minimum) required for tracking</li> <li>Marker tracking frequency: &gt;25 Hz</li> <li>Compensation range: Spherical space of diameter 50 mm</li> <li>Acquisition: accomplished by three-1D charge coupled device (CCD) sensors</li> </ul>
Accuracy		• Targeting centroid accuracy better than 1.5 mm <sup>10</sup>
Communication		Standard Treatment Couch – Serial RS232 cable     RoboCouch® System – Serial RS232 Infrared port
Operating Range		<ul> <li>Camera Array – vertical positioning approximately 2 m from the floor</li> <li>Camera Array in line with patient</li> </ul>
	Modes	<ul> <li>Manual</li> <li>User-Defined</li> <li>Automatic, system assisted</li> <li>Single-click, fully automated model building</li> </ul>
Modeling	Types	Linear Curvilinear Bi-Curvilinear (The optimal model type that achieves the minimum overall correlation error is automatically chosen)
	Error	Maximum allowable correlation error: 5 mm
User Displays (Selected)		<ul> <li>Breathing model/tracking markers waveform</li> <li>Algorithm parameters</li> <li>Correlation graphs</li> <li>Correlation error</li> <li>Error checking</li> </ul>
Hardware		<ul> <li>Single patient use custom Synchrony vest (3 sizes: S,M,L)</li> <li>Single patient use fiber optic markers with harness</li> </ul>

#### **4.2** INTEMPO™ ADAPTIVE IMAGING SYSTEM

The InTempo™ System is a time-based, motion tracking technology used to compensate for non-perodic intrafraction motion of the target. The InTempo System has been designed specifically to account for the types of motion patterns typically encountered when delivering radiation to the prostate and can only be used to deliver treatment to the prostate.

#### **Image Age**

The InTempo System makes use of the Image Age parameter to control treatment beam delivery. Image Age is the time elapsed since the most recent image acquisition. With the InTempo System, the system modifies guidance imaging parameters to ensure that no treatment beam is delivered based on an image that is older than a user-specified value.

#### **Adaptive Imaging**

With the InTempo System, the user may optionally allow the system to trigger adaptive imaging in the event that the assessed rate of target motion is greater than a user-defined threshold. Breach of this threshold causes the system to automatically reduce the maximum allowable image age. The user may also optionally choose to allow the system to continue treatment delivery at this increased resolution without triggering an E-stop.

<sup>&</sup>lt;sup>10</sup>Data on file.



Table 11: InTempo System Features

InTempo™ Adaptive Ir	naging System Features
General	<ul> <li>Time-based imaging</li> <li>Automatic and adaptive imaging based on detected target motion</li> <li>Graphical, time-based display of target position</li> <li>Graphical, time-based display of target orientation</li> </ul>
User Accessible Features	<ul> <li>Allowable Image Age (15 – 150 seconds)</li> <li>Target excursion thresholds for translational (mm) and rotational (deg) target shifts</li> <li>System response on target shift beyond any one of user-specified thresholds         <ul> <li>Adaptive Imaging triggered</li> <li>Pause Treatment</li> <li>No Action</li> </ul> </li> <li>Fiducial tracking algorithm parameters</li> </ul>
Requisites	<ul> <li>Compact generators</li> <li>Oil cooled X-ray sources</li> <li>Fiducial tracking</li> <li>CyberKnife® System software version 8.5 or higher</li> <li>MultiPlan® Treatment Planning System version 3.5 or higher</li> <li>CyberKnife Data Management System version 1.5 or higher</li> </ul>
Targeting Accuracy	• See Table 9

# 4.3 FIDUCIAL TRACKING

For lesions that are located extra-cranially, target tracking can be carried out with the use of fiducials. The system specifications are detailed in Table 12.

**Table 12: Fiducial Tracking Specifications** 

Fiducial Tracking Specifications				
General	<ul> <li>Minimum 3 fiducials required for accurate target tracking, including corrections for both translations and rotations</li> </ul>			
Output to User	<ul><li>Fiducial Location(s)</li><li>Probability measure value for extracted fiducial combination</li></ul>			
User Accessible Features	<ul> <li>Rigid body distance threshold (mm)</li> <li>Fiducial spacing threshold (mm)</li> <li>Collinearity threshold (deg)</li> <li>X-axis pairing tolerance (mm)</li> <li>Confidence threshold (%)</li> <li>Tracking range (mm)</li> </ul>			
Targeting Accuracy	• See Table 9			

The list of fiducials that may be used with the CyberKnife System is provided in the following section.



#### FIDUCIALS APPROVED FOR SOFT TISSUE TRACKING 11

The fiducials listed below have been validated for use with the CyberKnife® System for purposes of soft tissue tracking.

Table 13: Approved Fiducials for Soft Tissue Tracking

Manufacturer	Туре	Manufacturer's Part Number <sup>12</sup>	Dimensions	Description	
	Gold Seed	MT-NW-887-808	0.8 x 3 mm	3 sterile soft tissue gold markers	
		MT-NW-887-809	0.9 x 3 mm	3 sterile soft tissue gold markers	
		MT-NW-887-812	MT-NW-887-812 1.2 x 3 mm		
CIVCO	CyberMark™ Gold Seed	MT-NW-887-851 (-853)	1.0 x 5 mm	Sterile placement needles (single or 3-pack) with 1 x 5 mm soft tissue marker	
	Coupled™ Marker	MT-NW-887-710	1.0 x 3 mm	Sterile placement needle with two 3mm	
		MT-NW-887-709	0.8 x 3 mm	gold soft tissue markers attached at a fixed	
	Visicoil™ coiled gold markers	FP-0755	0.75 x 5 mm	Coiled gold soft tissue marker on a carrier for transport, and easy to deploy into most needle hubs	
RADIOMED		VC-075-005-PL		Coiled gold soft tissue marker in a preloaded needle with bone wax at the tip	
		VC-010-005-PL	1.1 x 5 mm	Coiled gold soft tissue marker in a preloaded needle with bone wax at the tip	
OLYMPUS	Gold Sphere	- N/A -	Dia 1.5 mm	Individual gold spheres	
ALPHA OMEGA	Gold Seed	SMG0242-025	0.8 x 5 mm	25 gold seeds per vial, packaged non-sterile. 19G thin walled biopsy needle 10-11 cm or 15 cm length	

# FIDUCIALS APPROVED FOR IMPLANTATION IN BONE OR SPINE

The fiducials listed below have been validated for use with the CyberKnife System for purposes of implantation in, and tracking of bony structures.

Table 14: Approved Fiducials for Bone or Spine Implantation

Manufacturer	Туре	Manufacturer's Part Number	Dimensions	Description	
BIOMET FIXATION (formerly Lorenz Surgical)	Stainless Steel Screw	31-6285	2.0 x 5.0 mm	Self drilling marker for use in spine or bone, 10-pack	
	Stainless Steel Screw	019005		Single self drilling marker per pack	
ACCURAY		019006	2.0 x 5.0 mm	Five (5) self drilling markers per pack	
		019007		Ten (10) self drilling markers per pack	
ACCURAY	Fiducial in- strument kit	018985	-N/A-		

<sup>&</sup>lt;sup>11</sup>Not all listed fiducial types may be available in all territories. Please contact the manufacturer for availability in your territory. <sup>12</sup>The manufacturer's part numbers listed above are valid as of March 2009. Please contact the manufacturer for equivalent part numbers in the event of obsolescence.



#### 4.4 XSIGHT® SPINE TRACKING SYSTEM

The Xsight® Spine Tracking System enables the tracking of skeletal structures in the cervical, thoracic, lumbar and sacral regions of the spine for accurate patient positioning and radiation beam delivery using the CyberKnife® System without implanting fiducials.

Target tracking with the Xsight Spine System is accomplished using 2D-3D registrations on a hierarchical mesh where local displacements at each of the mesh points are estimated and combined to provide 6D corrections to the treatment manipulator. The treatment manipulator uses these corrections to automatically deliver radiation to the displaced position of the target.

Table 15: Xsight Spine Tracking System Features

Xsight Spine Tracking System Features				
Tracking Mode	Direct tracking of spine lesions without the use of fiducials			
Eligibile Regions	Cervical, Thoracic, Sacral and Lumbar			
User Accessible Features	<ul> <li>Target dxAB threshold (mm)</li> <li>Target drAB threshold (deg)</li> <li>False nodes threshold (%)</li> <li>ROI height (mm)</li> <li>Tracking range in X and Y (mm)</li> <li>Confidence threshold (%)</li> <li>Live image contrast factor (automatic; independent manual override for Image A and B)</li> </ul>			
Targeting Accuracy	• See Table 9			

#### 4.5 XSIGHT LUNG TRACKING SYSTEM

The Xsight Lung Tracking System tracks tumors in the lung directly and without the use of fiducials, by using image intensity differences between the lesion and the background. The Xsight Lung System works in conjunction with the Xsight Spine Tracking System to track translational motions of the lesion. Patient alignment is accomplished with the use of the Xsight Spine System's spinal segmentation capabilities while during the treatment Xsight Lung Tracking System tracks the translational motion of the tumor.

Table 16: Xsight Lung Tracking System Features

Xsight Lung Tracking System Features				
Tracking Mode	Direct tracking of lung lesions without the use of fiducials			
Lesion Eligibility	<ul> <li>Minimum 15 mm across all dimensions</li> <li>Located in the peripheral lung region</li> <li>Visible in both live X-ray projections and both DRR images</li> </ul>			
Requisites	Xsight Spine Tracking System     Synchrony® Respiratory Tracking System			
User Accessible Features	<ul> <li>Target dxAB threshold (mm)</li> <li>Tracking range in X (mm)</li> <li>Tracking range in Y (mm)</li> <li>Confidence threshold (%)</li> <li>Live contrast factor</li> <li>Preferred Projection (ON/OFF)</li> <li>Tumor Region (ON/OFF)</li> <li>In Plane Rotation (ON/OFF)</li> <li>Live image contrast factor (automatic,; independent manual override for Image A and B)</li> </ul>			
Targeting Accuracy	• See Table 9			



#### 4.6 6D SKULL TRACKING

The 6D Skull Tracking feature in the CyberKnife® System allows direct and non-invasive tracking of intracranial lesions. Target tracking and motion compensation are accomplished by identifying and tracking rigid skull anatomy by using image intensity and brightness gradients between the DRR and live images. Patient setup, alignment and lesion tracking is done non-invasively and without the use of rigid head mounted frames.

Table 17: 6D Skull Tracking System Features

6D Skull Tracking System Features				
Tracking Mode	Direct tracking of intracranial lesions without the use of fiducials			
User Accessible Features	Brightness window (%): Enabled/Disabled     Gradient window (%): Enabled/Disabled			
Targeting Accuracy	• See Table 9			

#### 4.7 SUPPORTED TRACKING COMBINATIONS

#### Table 18: Selectable (YES) and Non-Selectable (NO) Tracking Mode/Patient Position Combinations

	SUPINE		PRONE	
Tracking Mode	Head First (HFS)	Feet First (FFS)	Head First (HFP)	Feet First (FFP)
Fiducial Tracking	Yes	Yes	Yes	Yes
Fiducial Tracking with Synchrony® Respiratory Tracking System	Yes	Yes	Yes	Yes
Fiducial Tracking with InTempo™ Adaptive Imaging System <sup>13</sup>	Yes	Yes	Yes	Yes
6D Skull Tracking	Yes	No	No	No
Xsight® Spine Tracking	Yes	Yes	No	No
Xsight Lung Tracking with Synchrony Respiratory Tracking System	Yes	Yes	No	No

## 5. TREATMENT PLANNING

The MultiPlan® Treatment Planning System is a dedicated planning system for use with the CyberKnife Robotic Radiosurgery System. The MultiPlan System is provided with both hardware and software.

The MultiPlan System provides the tools necessary to perform a complete range of treatment planning tasks, from image registration, target and critical structure delineation, through dose optimization, calculation and plan review. The planning system uses measured beam data imported into the Data Management System to accurately calculate dose in a patient's treatment plan.

Several updates have been made to the MultiPlan System as part of Accuray's continuing effort to respond to customer feedback. In addition to these updates, purchasable options are available that add further functionality to the MultiPlan System.

#### 5.1 HARDWARE SPECIFICATION

New MultiPlan Systems include the following hardware configurations:

- Windows Server 64-bit OS
- Dual quad core 3Ghz processor
- 24GB memory
- 512 MB Graphics card

<sup>&</sup>lt;sup>13</sup>The use of the InTempo System is restricted to treatment delivery to the prostate.

#### 5.2 MULTIPLAN® SYSTEM SOFTWARE

Depending on your MultiPlan® System, software may include the following features and enhancements:

- AutoSegmentation<sup>TM</sup> option (Automatic segmentation for prostate and associated VOIs.)
- QuickPlan<sup>TM</sup> option (Automation of the plan generation process.)
- Treatment time reduction feature. (Smart reduction of treatment time. Enables Robotic IMRT™)
- Advanced Visualization (Enhanced 3D rendering of image data, VOIs and dose.)
- Support for colormap PET image data
- Enhanced plan QA tools
- Improved dose memory handling
- Support for the CDMS reporting package
- Medium resolution dose calculation
- Support for Accuray Treatment Planning Services (ATPS)
- New computation platform (Dell T7500)
- Monte Carlo Dose Calculation at CT resolution

#### 5.3 DISPLAY LAYOUTS AND CONTROLS

The display of images and treatment plans in the MultiPlan System are controlled by the user. Screen layouts can be modified as needed, and individual images can be manipulated using the controls provided.

Layouts include the following:

- 2D axial, sagittal and coronal views
- 3D rendering of the CT
- DVH (Dose Volume Histogram)
- Plan and Dose Statistics
- Side-by-side 3D view
- Full screen single view
- Advanced Visualization allows additional layouts to be selected and allows layout to be changed at any point in the planning process

Image controls include the following:

- Window and level controls
- Zoom capability
- Pan capability
- Isocontour selection
- Absolute or Relative dose display
- Maximum dose display
- Uncertainty colorwash display (only with the Monte Carlo Dose Calculation feature)



# 5.4 USER PRIVILEGE AND SECURITY SETTINGS

User Privileges for the MultiPlan System are defined in the CyberKnife® Data Management System. Assignable privileges include:

- Load Treatment Plan
- Create Treatment Plan
- Create QA Plan
- Save Deliverable Treatment Plan

- Delete Treatment Plan
- Access Commissioning Tools
- Approve Commissioning Data
- Access Preferences and Plan Templates

#### 5.5 FUSE (IMAGE REGISTRATION)

The MultiPlan System allows the user to register the following imaging modalities to the primary  $\operatorname{\mathsf{CT}}$ .

- CT
- 4D CT (only with the 4D Treatment Optimization and Planning feature)
- MR (acquired in either the axial, sagittal, coronal or oblique (up to 30°) planes)
- PET (Colormap and grayscale)
- XA (3DRA)

Registration of images can be performed using any (or a combination) of the following options:

- Automatic registration using intensity values based on mutual information
  - The user can choose to select a Fusion Region of Interest when fusing automatically
- Semiautomatic seed point registration allowing user selection of up to 8 seed points
- Manual registration with fine adjustment tools
- For 4D CT only, deformable registration using intensity values and user defined seed points





If either the Automatic or Semiautomatic registration option is selected, the MultiPlan® System provides real-time visual feedback of the registration during the process.

Feedback is displayed in the following views:

- Split view
- Checkerboard view
- Merged view

If multiple secondary image series are to be registered to the primary CT, and if those secondary image series are already registered to each other, then the MultiPlan System provides the option to register one secondary image series to the primary CT and then copy that registration information to the other secondary image series.

#### 5.6 CONTOUR (DELINEATION)

The MultiPlan System provides the ability to delineate many types of contours for treatment planning purposes.



#### 5.6.1 VOI (Volume of Interest)

Volumes of Interest include:

- Targets anatomic volume where dose is prescribed
- Critical structures anatomic volume where dose is constrained
- Shell structures planning volume that surrounds the target and is constrained to increase the gradient outside the target volume
- Tuning structure planning volume that is constrained to further minimize dose to healthy tissue
- Cavity anatomic area encompassed by and subtracted from either a target or a critical structure

User options for delineation of VOI include:

- Delineation of up to 22 Volumes of Interest (VOI), with any mix of targets and critical structures
- Delineation of up to 32 contour sets for each VOI, to define non-contiguous segment of the same structure in each slice
- Delineation on axial, sagittal or coronal planes
- AutoSegmentation™ for Prostate (optional, See section 5.18)
- 2D and 3D delineation tools include:
  - Pen or drawing tool
  - Line segment tool
  - Ellipse tool
  - 2D Magic Wand uses high contrast to automatically delineate a VOI on a single image slice
  - 3D Magic Wand uses high contrast to automatically delineate a VOI on all image slices
  - Rescale tool move or re-size a drawn delineation
  - Smart Curve adjusts a user drawn delineation to match a border identified by high contrast
  - Bumper Tool nudge a drawn delineation in any direction
  - Auto interpolation interpolates delineations on image slices between two image slices that have a drawn delineation
  - 3D copy copy all the drawn and interpolated contour sets into another contour set
  - 3D dilation/erosion 3D isotropic or anisotropic expansion or contraction of a critical or target volume

# Properties Template Select Contour Set New Delete Conviry Auto interpolation Drawing Tools Undo Redo Delete All Delete 3D Copy Dilate Copy Paste Options Simart curve fitting Display all VOIs Display all VOIs

#### 5.6.2 Skin

Provides automatic delineation of the external border of the patient anatomy. The skin structure defines the boundary between where dose is displayed (inside the skin) and where dose is not displayed (outside the skin).

#### 5.6.3 Spine Tracking Volume

The Spine Tracking Volume defines the gross extent of the spine for use with Xsight® Spine Tracking System and Xsight Lung Tracking System. During treatment delivery the delineated Spine Tracking Volume can be displayed or suppressed on the DRR (Digitally Reconstructed Image) display to assist with visualization of the target.

#### 5.6.4 Ball-cube

Provides automatic delineation of a sphere, such as in the Ball-cube Quality Assurance (QA) tool to assist with treatment planning for QA tasks such as end-to-end testing.



#### 5.7 TREATMENT PLANNING

#### 5.7.1 Isocentric Planning

Two planning options are available in the Isocentric Planning feature of the MultiPlan® System.

- · Forward planning provides the option to generate a set of beams that are isocentrically targeted and evenly weighted
- Inverse planning provides the option to generate a set of beams that are isocentrically targeted and non-evenly weighted

#### 5.7.2 Conformal Planning

Conformal planning produces a plan by optimizing the beam weights for a given beam set. In conformal planning, beams are aimed at randomly selected points on the target boundary surface and then the dose distribution is optimized by adjusting the weighting (number of MUs) of the beams according to user-set dose constraints.

- Simplex optimization algorithm minimizes the total number of MU for the plan as well as the deviations above the maximum dose constraints and below the minimum dose constraints
- Iterative optimization algorithm optimizes deviations above the maximum dose constraints and below the minimum dose constraints
- Sequential Optimization algorithm see section 5.13 for details

#### 5.7.3 Dose Calculation Algorithms

# 5.7.3.1 Ray Tracing Dose Calculation

The ray-tracing dose calculation algorithm uses three system-specific beam description tables comprised of data measured using a water phantom. The tables are Tissue Phantom Ratio (TPR), Off Center Ratios (OCR) and Output Factor (OF).

- TPR values for each collimator are normalized to a value of 1.0 at a depth of 15 mm
- OCR values for each collimator are normalized to a value of 1.0 at a radius of 0 mm
- OF value for each collimator is normalized to the value of the OF for the 60 mm collimator at 800 mm SAD and 15mm depth, which is defined to be 1.0

Tissue density corrections can be applied using the effective path length algorithm. Surface obliquity can be corrected using a geometric ray tracing algorithm.

#### 5.7.3.2 Monte Carlo Dose Calculation – see section 5.11 for details

#### 5.8 EVALUATION

#### 5.8.1 Plan Statistics

The following statistics are available for each plan generated in the MultiPlan System.

- Dose Volume Histograms (DVH) can be displayed for each VOI
- Minimum Dose in centiGray (cGy) for each VOI
- Maximum Dose in cGy for each VOI
- Mean Dose in cGy for each VOI
- Conformality Index (CI) The ratio of tissue volume that receives the prescription isodose or more to tumor volume that receives the prescription isodose or more
- New Conformality Index (nCl) the CI multiplied by the ratio of the total tumor volume to the tumor volume that receives the prescription isodose or more
- Homogeneity Index (HI) the ratio of the maximum dose to the prescription dose
- Coverage the volume of the target that receives the prescription dose divided by the total volume of the target

#### 5.8.2 Isodose Curves

Once a plan is generated, isodose curves are displayed on every 2D view.

Users can configure this display as follows:

- Line color and line thickness can be adjusted
- Display either Absolute Dose or Relative Dose
- Display or hide each curve individually
- Display or hide minimum and maximum dose points for all VOIs



#### 5.8.3 3D Rendering

View rendering of patient beams, delineated structures and calculated dose in 3D. MultiPlan® has the capability to render image data, VOIs and dose in 3D. Image data can be rendered in 3D at native resolution. The user is provided with anatomic specific filters for image data rendered in 3D. The user also has the ability to create and save custom filters. Layout of 2D and 3D images is customizable. Advanced Visualization tools are available at any point in the planning workflow

#### 5.9 OA TOOLS

#### 5.9.1 Plan Comparison

Using the Reference Plan feature, compare two treatment plans for the same patient (using the same CT series) side-by-side to determine the best plan for the patient.

# 5.9.2 Plan Summation

Using the Sum Plans feature, add dose from two or more plans (using the same CT series) together.

#### 5.9.3 Beam List

View a list of all beams and parameters in the selected treatment plan. Parameters displayed include:

- Beam number
- Template Path ID / Node ID
- Collimator size
- Source to Axis Distance (SAD)
- Depth (cax) euclidean measure of depth along the central axis of the beam
- Depth (effective) equivalent depth through water of the beam
- Radius at 800 mm
- OCR \* INV<sup>2</sup> \* TPR \* dm

(Off Center Ratio x inverse square x Tissue Phantom Ratio x Output Factor)

- Monitor Units
- Dose (cGy) from the Ray Trace algorithm
- Dose (cGy) from the Monte Carlo Dose Calculation algorithm

#### 5.9.4 Animation

View a graphical simulation of the selected treatment plan, including robot motion. Animation includes a proximity detection simulator.

# 5.10 COMMISSIONING TOOLS


#### 5.10.1 Graphical Display

Once beam data is imported into the CyberKnife® System, it is displayed graphically in the MultiPlan System. Measured, calculated and reference beam data are displayed for the following:

- Off center ratio
- Tissue phantom ratio
- Output factor

User approval of beam data is required.



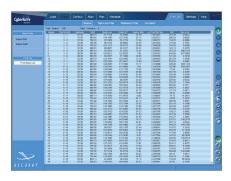






#### 5.11 MONTE CARLO DOSE CALCULATION<sup>14</sup> (OPTIONAL)

The Monte Carlo dose calculation algorithm samples the interactions of photons entering the patient. Sufficient photons are sampled to provide a statistically accurate calculation of the dose deposited in the patient. The algorithm includes a source model that describes the distribution of the energies and trajectories of photons exiting the linac . The patient geometry is modeled as 3D arrays of mass density and material type. Photons interact with the patient volume to form particles, such as electrons and positrons, and deposit energy. Each particle is tracked and, for each voxel, the deposition of its energy is calculated. The total dose recorded at each voxel is an estimate of the dose deposited by all photons in the actual treatment beam.

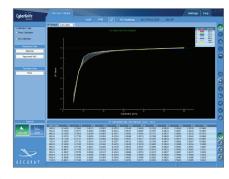


The Monte Carlo dose calculation algorithm carries out the necessary computations in a matter of minutes rather than hours without compromising dose calculation accuracy. The algorithm


uses a beam commissioning procedure to derive source model parameters automatically based on measured beam data. The Monte Carlo commissioning procedure requires several days of computation time; however it is only performed during the commissioning process and can be broken up into a few hours at a time. The Monte Carlo dose calculation can be enabled for both fixed collimators and the Iris™ Variable Aperture Collimator.

# Prerequisites:

- The system has both Monte Carlo and Iris licenses installed
- Monte Carlo commissioning for Iris has been performed


If the user selects the Iris Collimator in the Treatment Parameter of the Align tab, Monte Carlo will be available as a dose calculation method. (See figures below)

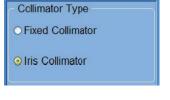

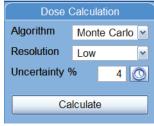

In addition to enabling Monte Carlo dose calculation for the Iris Variable Aperture Collimator, the following Monte Carlo enhancements are included in v3.5 of the MultiPlan System.

- Monte Carlo In Air Output Factor measurement is optional
- The user can fit a Gaussian Source Distribution both for Fixed and Iris Collimators in the Monte Carlo commissioning procedure
- FWHM of the Gaussian source distribution can be adjusted to change the penumbra slope of the Monte Carlo OCR curves to match the measured OCR curves

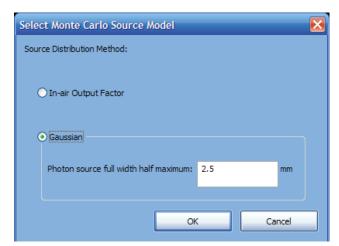

Dialog to choose between In-Air Output Factor, or Gaussian source model.

<sup>&</sup>lt;sup>14</sup>See Monte Carlo Dose Calculation white paper Accuray P/N 500317 for additional technical specifications.



#### 5.12 4D TREATMENT OPTIMIZATION AND PLANNING (OPTIONAL)

4D Treatment Optimization and Planning utilizes 4D CT scans for treatment planning. Target-centric alignment of all the phases of the 4D CT provides a 'beams eye view' of the motion and deformation of not only the target but also the surrounding critical structures. The planning optimization takes this motion and deformation into account when calculating dose to these volumes of interest.

Treatment plans generated using the 4D Treatment Optimization and Planning option can be delivered using the Synchrony® Respiratory Tracking System in combination with either Fiducial Tracking or Xsight® Lung Tracking System.

To ensure compatibility of each individual 4D CT scanner with the MultiPlan® Treatment Planning System, a sample image set isrequired by Accuray.

#### 5.13 SEQUENTIAL OPTIMIZATION (OPTIONAL)

Treatment planning with Sequential Optimization begins with a series of user-defined clinical objectives as input. Examples of clinical objectives may be:

- For targets
  - Dose maximization
  - Conformality
  - Homogeneity
  - Coverage
- For critical structures
  - Dose minimization
- For the whole treatment plan
  - Total number of monitor units (MU) delivered

Clinical objectives established for each target and critical structure are sequenced together to make up steps in a script and are optimized one at a time until every objective is optimized.

Enhancements to Sequential Optimization for the MultiPlan System v3.5 include:

- Beam reduction enhancements
- Relaxed convergence
- Dose statistics for shell structures
- Node Reduction
- Treatment time reduction (see section 5.19)

# **Beam Reduction Enhancements**

In the MultiPlan System v3.0 / 3.1, the beam reduction dialog box only had an OK and Cancel button. Clicking the OK button meant performing a beam reduction immediately followed by restarting optimization.

In this release we introduce another beam reduction option, which is


to perform beam reduction only followed by a dose calculation.

As shown in the dialog below, the OK button has been replaced with the Reduce and Optimize button while a Reduce button has been added to allow the user to perform a beam reduction only.

# Relaxed Convergence

When the Relaxed Convergence box is checked, a set of predetermined parameters are activated which will cause optimization of a particular objective to stop if it is within a certain range of the specified goal. Optimization of this step will complete and the Sequential Optimization algorithm will proceed to the next step.

#### Beam reduction dialog box for the MultiPlan System v3.5.

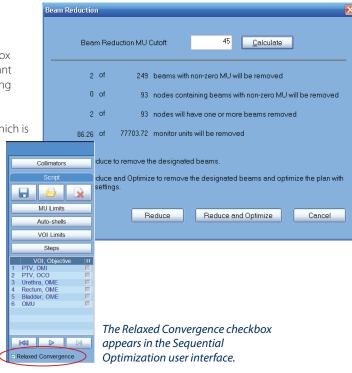



#### **Dose Statistics for Shell Structures**

Dose Statistics Table							
VOI	Min (cGy)	Mean (cGy)	Max (cGy)	CI	nCI	HI	Coverage
Urethra	3564.37	3997.37	4475.91				
Prostate	3514.67	5050.93	7034.61	1.35	1.38	1.85	98.42%
■ PTV	3206.28	4927.11	7034.61	1.16	1.21	1.85	96.19%
Penis	75.57	722.40	3292.37				
Bladder	63.12	644.48	4483.54				
Rectum	89.95	1165.68	3698.98				
Seminal Vessels	2838.42	4054.74	5132.09	61.56	86.77	1.85	70.95%
Bowel	58.18	94.39	1384.92				
Pubic bones	281.77	1671.84	4688.13				
☐ PTV - Shell 1	2034.22	3038.55	4092.92				
☐ PTV - Shell 2	275.87	1247.63	1825.77				

Sequential Optimization Shell statistics are shown in the Dose Statistics table on all steps except for the Reference Plan step. Note: a hollow square indicates a shell.

#### 5.14 MULTIPLAN® MD SUITE (OPTIONAL)

Prior to version 3.5 of the MultiPlan® System, the tool usually used by the CyberKnife® physician for remote review of treatment plans, contouring, fusion, set treatment parameters, was the InView® Image Fusion and Contouring Station. Concurrent with the release of the MultiPlan System v3.5, is the availability of MultiPlan MD Suite as an option for remote review, contouring, fusion etc.

The primary difference between MultiPlan MD Suite and the InView station is that MD Suite is networked directly to the CyberKnife Data Management System. Therefore, any changes made to a patient directory while working in MD Suite, are immediately available to the MultiPlan user without the need for a manual data transfer.



MultiPlan MD Suite entry page.



MultiPlan MD Suite Plan task.



#### 5.15 MULTIPLAN QUICK REVIEW (OPTIONAL)

MultiPlan® Quick Review allows the user to run one primary and up to three secondary sessions of the MultiPlan System simultaneously. The primary session has full functionality. The secondary sessions cannot perform optimization. Sessions can be started through the desktop, or the session manager interface.

The session manager interface is accessible on the bottom right corner of the splash page, or on any subsequent screen during the planning process.

The Multiple Sessions feature allows the user to run one MultiPlan session along with three Quick Review sessions all at the same time.

The Quick Review sessions allow fusion, contouring, treatment plan review, change of prescription and dose calculation.




Secondary sessions can be either initiated through the MultiPlan icon on the desktop, from the start up menu, or from the session manager interface. A green button on the session manager interface indicates the active session. If the user places the mouse over an active session button, a tooltip will appear indicating the patient and plan name.

#### 5.16 DENSITY OVERRIDE

The density override feature allows the user to assign a density to any contoured VOI. Typically, this tool can be used to correct for an artifact produced by a metallic object, or, to assign a density to a region in the patient that was present at the time of scan, but not present at the time of treatment. e.g. contrast in the bladder.

The density override tool can be accessed in the Plan tab under the Setup menu.


When VOIs with assigned densities overlap, the density of the VOI higher on the list will be used in the region of intersection.

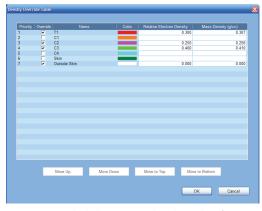

Density Override dialog. VOIs are listed in order of priority.


## 5.17 QUICKPLAN™ (OPTIONAL)

QuickPlan<sup>TM</sup> can completely automate all aspects of treatment plan generation. The user selects a QuickPlan template and starts the application. All steps of the plan generation process (fusion, fiducial location, contouring in the case of prostate, setting of plan parameters, optimization and dose calculation), proceed automatically. Note: fusion and AutoSegmentation<sup>TM</sup> for prostate require minimal user input. The user is presented with a completed treatment plan for review. Prior to saving a plan as deliverable, the user will be prompted to review all elements of the treatment plan that were generated automatically. For a detailed description of QuickPlan, please see the user's manual.

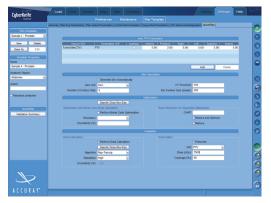

The QuickPlan tab.

# CyberKnife<sup>\*</sup>

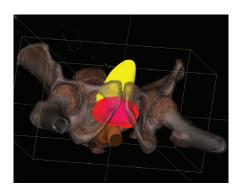


AutoSegmentation allows automatic contouring of prostate, rectum, bladder, seminal vesicles and femoral heads.

#### 5.18 AUTOSEGMENTATION™ (OPTIONAL)

AutoSegmentation™ allows automatic contouring of prostate, rectum, seminal vesicles, bladder and femoral heads. Minimal user input is required. Before running the algorithm, the user must provide initialization points for all structures except femoral heads. Initialization points can be placed on either CT or MR image data. 3D editing tools are provided if the results of the algorithm need to be altered. For a detailed description of AutoSegmentation and the protocol for placement of initialization points, please see the user's manual.

#### 5.19 TREATMENT TIME REDUCTION TOOL

MultiPlan® will calculate an estimated treatment time for all plans generated. The Treatment Time Reduction tool can be used to reduce estimated treatment time for both radiosurgery and conventionally fractionated treatment plans (Robotic IMRT<sup>TM</sup>). Once the initial optimization completes, the user can enter their desired treatment time. The Treatment Time Reduction tool also utilizes smart targeting algorithms when removing beams. For a detailed description of the Treatment Time Reduction Tool, please see the user's manual. The MultiPlan System will then generate a family of treatment plans by successive iterations of beam reductions, node reductions and re-optimizations. As the iterations proceed, the trend for the predicted treatment times will decrease until the user-specified treatment times is achieved.

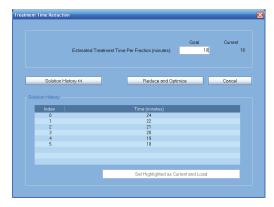

The treatment time reduction tool is accessed in the Sequential Optimization tab.

#### 5.20 IMPROVED PET SUPPORT

MultiPlan supports both grayscale and colormap PET images. These images can be displayed in 2D and 3D. The user is provided with three methods for SUV calculation (body weight, body surface area, lean body mass) or the value for SUV in the DICOM header can be used.



#### 6. DATA MANAGEMENT

#### 6.1 CYBERKNIFE® DATA MANAGEMENT SYSTEM

#### 6.1.1 Data Server

The data server hosts the CyberKnife® System database and processes requests received by the MultiPlan® System, the CyberKnife Treatment Delivery System, and the administration workstations.

The data server resides in a rack mounted PC in the equipment room.

#### Data server hardware specifications

- Dual processor with each processor having a clock speed > 2 GHz
- 4 GB of memory
- Set of SCSI 15K rpm disk drives in a RAID 5 configuration providing approximately 500 GB
- Set of SATA 7.2K rpm disk drives setup in a RAID 5 configuration providing approximately 1 TB of storage
- Two Gigabit Network Interface Cards (NIC)
- Redundant power supplies

#### 6.1.2 Administrative Workstation

The administration workstation houses data management applications and archived patient records. Applications on the workstation include:

- Patient Record Archive and Restore
- Image Review and Import
- Patient Administration
- DICOM Administration
- Plan Administration
- Beam Data Import
- User Administration
- System Administration
- Report Administration

# Administration Workstation specifications

- Single processor (3 GHz) or 2.3 GHz Dual Core
- > 1 GB of memory
- Set of SATA 7.2K rpm disk drives setup in a RAID 10 configuration providing approximately 1 TB of storage
- One gigabit NIC
- DVD Recorder capable of writing to a DVD with either DVD+R or DVD+R DL format
- 21" Flat Panel LCD Monitor

#### 6.2 HIPAA COMPLIANCE SUPPORT

The following are the policies enforced and capabilities provided by the CyberKnife Data Management System to help users implement internal Health Insurance Portability and Accountability Act of 1996 (HIPAA) compliance programs.

- Provides user profile management capabilities enabling a system administrator to limit actions available to a user account
- Provides user management policies enabling a system administrator to force a user to change his/her password every configurable amount of days
- Secures transfer of user login information over hospital network
- Records actions performed on a patient record by a user account



# 7. PATIENT POSITIONING SUPPORT

#### 7.1 TREATMENT COUCH


Two types of patient positioning support systems are available with the CyberKnife® System; the Standard Treatment Couch and the RoboCouch® Patient Positioning System (optional).


**Table 19: Patient Positioning System Specifications** 

	Standard Treatment Couch System	RoboCouch® Patient Positioning System	
Payload	159 kg (350 lb)	227 kg (500 lb)	
Range of Motion			
Anterior/Posterior	28 cm	42 cm	
Right/Left	± 15 cm	± 18 cm	
Superior/Inferior	≥ 91 cm	≥ 100 cm	
Head Up/Head Down (Pitch)	±5°	± 5°	
Right/Left Tilt (Roll)	±5°	±5°	
Yaw (CW/CCW)	N/A	±5°	
Control	Remote workstation     Local hand pendant	Remote workstation     Local hand pendant	
Repeatability			
Translational	0.3 mm	0.1 mm	
Rotational	0.3°	0.1°	
Motion Corrections	Most Degrees of Freedom are corrected serially	All Degrees of Freedom are corrected simultaneously	
Pont of Rotation	Fixed; determined by mechanical assembly of the actuators	Variable; all axes can move simultaneously about a set point in space	

#### 7.1.1 Standard Treatment Couch

The Standard Treatment Couch is the standard patient support system of the CyberKnife System. It provides the user with flexibility in patient positioning by providing 5-DOF motion capabilities.

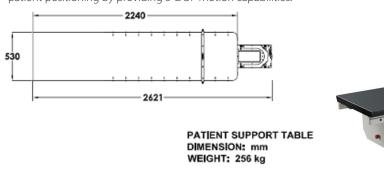

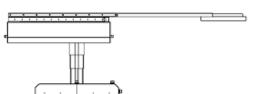






#### 7.1.2 RoboCouch® Patient Positioning System

The RoboCouch® System provides a highly flexible 6-DOF mechanism for automatically positioning the patient. The combination of the RoboCouch System and the robotic manipulator for linac positioning enables the CyberKnife® System to deliver dose precisely, and to the right location. The upper manipulator arm (between axes A2 and A3 as shown) integrates a contact sensor on its outer surface and an E-STOP is triggered if an object comes in contact with it. Both versions of the couch top are described in sections 7.2.1 and 7.2.2 respectively. The RoboCouch System is available with either a flat carbon fiber couch top (standard with the RoboCouch System) or a seated load carbon fiber table top (optional). The RoboCouch System has five rotational axes is shrouded in a free-standing cover.

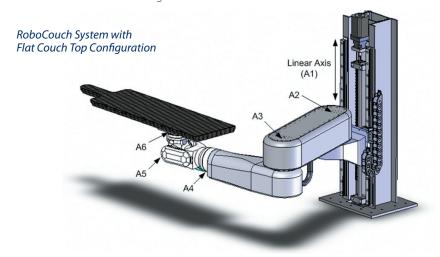

#### 7.2 TREATMENT TABLE TOP


There are three different tops available for use with CyberKnife System: flat top for the Standard Treatment Couch, flat top for the RoboCouch System and the Seated Load table top for the RoboCouch System.

# 7.2.1 Flat TableTop (Standard)

The flat table top included with the RoboCouch version of the patient support system is described in section 7.1.

## 7.2.2 Seated Load Table Top (Optional)

The RoboCouch System is also offered with an articulating couch top allowing patients to be loaded in a seated position. The seated load table top converts to a flat table top to offer flexibility in positioning and loading the patient. The patient can be treated either in a knees-flat or a knees-elevated position. Table top characteristics are highlighted in section 7.2.

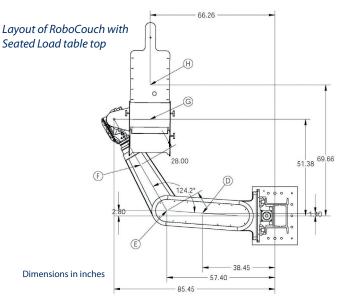



<sup>&</sup>lt;sup>15</sup>The Seated Load Table Top adds an additional rotational axis to the RoboCouch.



Table 20: Treatment Couch Top Specifications

	Flat with Standard Treatment Couch	Flat with RoboCouch® System	Seated Load Option with RoboCouch System		
Radiolucency	Maximum: < 1.1 mm Aluminum equivalence at 120 kVp for the length of at least 62 inches from the superior most point				
Immobilization (Compatibilty)	Alpha Cradle®     Vacuum Lock Bags     Thermoplastic masks				
Indexing	Compatible with Civco Prodigy™ indexing systems				
Material	<ul> <li>Carbon fiber with structural foam core</li> <li>Color: Black</li> <li>Carbon fiber with structural foam core</li> <li>Color: Gray</li> </ul>				
Minimum Load Height	• ≤ 64 cm (25 in)	• ≤ 56 cm (22 in)	• ≤ 45 cm (17.9 in) (seated load) • ≤ 58 cm (23 in) (horizontal load)		
Dimensions	<ul><li>Length:213 cm (84 in)</li><li>Width: 53 cm (21 in)</li><li>Thickness: 7.6 cm (3 in)</li></ul>	<ul><li>Length: 206 cm (81 in)</li><li>Width: 53 cm (21 in)</li><li>Thickness: 5.7 cm (2.25 in)</li></ul>	<ul><li>Length: 212 cm (83.4 in)</li><li>Width*: 53 cm (21 in)</li><li>Thickness: 6.4 cm (2.5 in)</li></ul>		

<sup>\*</sup>Maximum width as measured edge-to-edge at the hand cushion is 82 cm (32.4 in)



# 8. REGULATORY CLASSIFICATION

The CyberKnife® System is classified as follows:

- Protection against electric shock: Class I, permanently connected
- Applied part: Patient treatment table only, Type B
- Protection against harmful ingress of water: IPXO no protection against ingress of water
- Methods of sterilization or disinfection: Not required
- Degree of safety in the presence of flammable mixtures: Not suitable for use in the presence of flammable mixtures
- Mode of operation: Continuous



Accuray Worldwide Headquarters

1310 Chesapeake Terrace Sunnyvale, CA 94089 USA Tel: +1.408.716.4600 Toll Free: 1.888.522.3740, ext 4337 Fax: +1.408.716.4601 Email: sales@accuray.com

Accuray Europe

Tour Atlantique 25e 1 Place de la Pyramide 92911 Paris La Défense Cedex France Tel: +33.1.55.23.20.20

Fax: +33.1.55.23.20.39

Accuray Asia Ltd.

Suites 1702-1704, Tower 6 The Gateway, Harbour City 9 Canton Road, T.S.T. Hong Kong Tel: +852.2247.8688 Fax: +852.2175.5799

Accuray Japan K.K.

Daini Tekko Building 6F 1-8-2 Marunouchi, Chiyoda-ku Tokyo 100-0005 Japan Tel: +81.3.6269.9556 Fax: +81.3.3217.0337